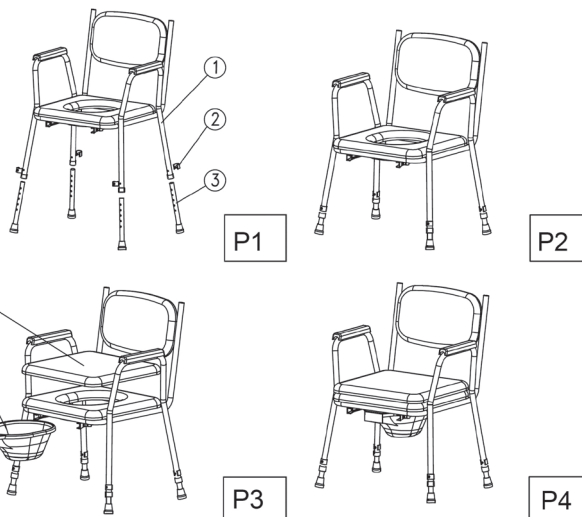
► SILLA INODORO DORMITORIO

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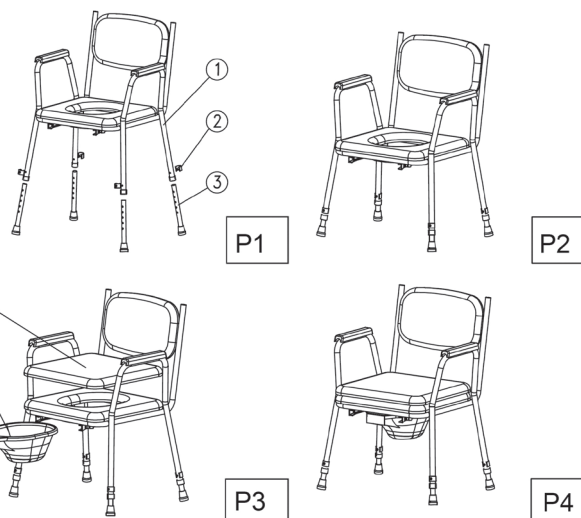
Dear Customer,

Thank you very much for your trust in an Actius by Orliman product. You have purchased a quality product. Please read the instructions carefully before using the product and use the product only as described and for the intended purposes.

If you have any questions about the use, maintenance or safety of your product, contact the healthcare establishment where you purchased the product or our customer service department. Keep these instructions for future reference.

REGULATIONS

This product is compliant with the European Directive on Medical Devices 93/42/EEC (Spanish Royal Decree 1591/2009). A Risk Analysis (EN ISO 14971) was done, minimising all existing risks.

Strictly observe municipal ordinance when discarding the packaging and product.

SUITABLE FOR

Recommended for patients with mobility or incontinence problems. It's quite convenient for users as they don't have to go all the way to the toilet. The occupant must not weigh more than the maximum weight indicated.

ASSEMBLY AND INSTALLATION

Step 1: As shown in image P1, mount the legs on the structure and adjust the height. Secure the screw. Complete as in P2.

Step 2: As in P3, place the commode pan under the seat and put the seat cover on. Complete as shown in figure P4.

PRECAUTIONS

The construction material is inflammable. Do not expose the products to situations that could set them on fire. In the event of a fire, quickly let go of them and use the proper resources to put it out. Do not expose the product to direct heat sources such as stoves, heaters, radiators, etc.

SAFETY AND MAINTENANCE

1. Check all nuts. Make sure they are all well-secured.

(2) Clean often and disinfect regularly.

3. Keep in a dry and well-ventilated place.

Cleaning: Clean and dry periodically. Remove any dirt with a damp cloth and neutral soap, as necessary. Then dry. Do not use any aggressive or abrasive chemical cleaners. They may damage the product material.

If contaminated with infected material, use standard disinfection resources.

WARRANTY

ORLIMAN, S.L.U. guarantees all its products as long as the original configuration has not been manipulated or altered. It does not guarantee any products with altered characteristics due to improper use, defects or breakage of any kind. The product is covered by a two-year warranty for material or manufacturing defects. Excluded from the warranty are imperfections caused by improper use, accidents or natural wear of parts such as plastic components, footrests, upholstery, brakes, tips, inner tubes and wheelchair tyres and similar parts, depending on the type of medical product acquired.

If you do not follow the instructions, warnings and the notes in the user's manual, you or a third party may be physically injured or the product may be damaged.

If you observe any defect, anomaly or failure, notify the dealer immediately so it may be replaced or repaired.

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