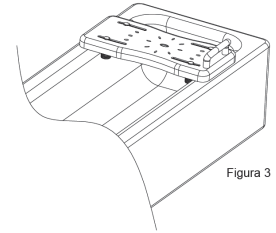
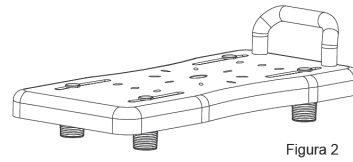
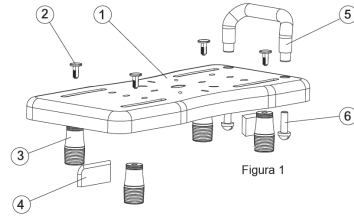
► TABLA DE BAÑERA

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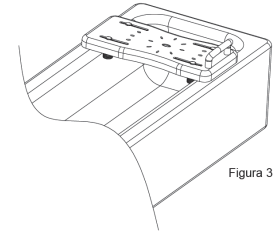
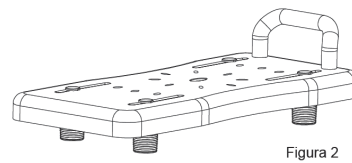
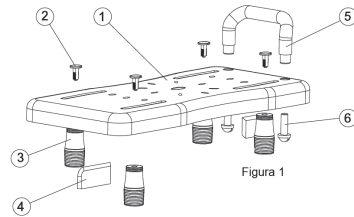
► BATHBOARD

Instructions for use and care

1. Bathboard
2. Plastic screw
3. Anti-slip rubber feet
4. Wedge
5. Handle
6. Handle screw

	ACAB02#UNI
Depth	320 mm
Width	710 mm
Height	190 mm
Maximum occupant weight	90 kg

Dear Customer,

Thank you very much for your trust in an Actius by Orliman product. You have purchased a quality product. Please read the instructions carefully before using the product and use the product only as described and for the intended purposes.

If you have any questions about the use, maintenance or safety of your product, contact the healthcare establishment where you purchased the product or our customer service department. Keep these instructions for future reference.

REGULATIONS

This product is compliant with the European Directive on Medical Devices 93/42/EEC (Spanish Royal Decree 1591/2009). A Risk Analysis (EN ISO 14971) was done, minimising all existing risks.

Strictly observe municipal ordinance when discarding the packaging and product.

SUITABLE FOR

Enables a lateral transfer from a wheelchair for easier bathtub entry for people with reduced mobility or a motor disability. The occupant must not weigh more than the maximum weight indicated.

ASSEMBLY AND INSTALLATION

Step 1: Assemble the bathboard as shown in figure 1 and figure 2.

Step 2: The 4 rubber feet must be adjusted to the sides of the bathtub so

the board is fixed and stable. To do so, loosen the 4 plastic screws and adjust the rubber feet to the sides of the bathtub. Once the 4 rubber feet are adjusted, tighten the plastic screws until securely fixed.

PRECAUTIONS

The construction material is inflammable. Do not expose the products to situations that could set them on fire. In the event of a fire, quickly let go of them and use the proper resources to put it out. Do not expose the product to direct heat sources such as stoves, heaters, radiators, etc.

RECOMMENDATIONS AND WARNINGS

- Carefully read these instructions before the user uses the chair or bench.
- The bath chair or bench is for sitting down in the shower or bathtub. It must only be used over showers or bathtubs with stable, non-skid floors. This product is designed for private use only and is not suitable for hospital use.
- This technical assistance must not be used for any other purpose except as intended. Do not use this product as a step stool. Do not stand on it. Do not use on furnishings. Do not allow children to play with this device.
- Do not modify the product or add other elements.
- Inspect periodically and before using to ensure the fixing elements are firmly secure. Carefully check the conditions of all components. If any are not in perfect conditions of use, they must be replaced with an original spare part.
- The skin in contact with pressure support areas must not be wounded or hyper-sensitive. Check for any redness on the skin, pressure or patient discomfort. Do not use in cases of irritation, pressure, pain or obvious discomfort.

SAFETY AND MAINTENANCE

Before using, make sure the screw is tight and the frame fits the user.

Cleaning: Clean and dry periodically. Remove any dirt with a damp cloth and neutral soap, as necessary. Then dry. Do not use any aggressive or abrasive chemical cleaners. They may damage the product material.

If contaminated with infected material, use standard disinfection resources.

WARRANTY

ORLIMAN, S.L.U. guarantees all its products as long as the original configuration has not been manipulated or altered. It does not guarantee any products with altered characteristics due to improper use, defects or breakage of any kind. The product is covered by a two-year warranty for material or manufacturing defects. Excluded from the warranty are imperfections caused by improper use, accidents or natural wear of parts such as plastic components, footrests, upholstery, brakes, tips, inner tubes and wheelchair tyres and similar parts, depending on the type of medical product acquired.

If you do not follow the instructions, warnings and the notes in the user's manual, you or a third party may be physically injured or the product may be damaged.

If you observe any defect, anomaly or failure, notify the dealer immediately so it may be replaced or repaired.

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