► ELEVADORES

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ACAB07#UNI

Dear Customer,

Thank you very much for your trust in an Actius by Orliman product. You have purchased a quality product. Please read the instructions carefully before using the product and use the product only as described and for the intended purposes.

If you have any questions about the use, maintenance or safety of your product, contact the healthcare establishment where you purchased the product or our customer service department. Keep these instructions for future reference.

REGULATIONS

This product is compliant with the European Directive on Medical Devices 93/42/EEC (Spanish Royal Decree 1591/2009). A Risk Analysis (EN ISO 14971) was done, minimising all existing risks.

Strictly observe municipal ordinance when discarding the packaging and product.

SUITABLE FOR

Makes it easier for trauma patients to sit on or stand up from the toilet. The lift is anatomically shaped and features fixing elements for a better hold and balance. It meets the needs of people with some type of physical disability by giving users the support they need to stand up from or sit down on a toilet. The occupant must not weigh more than the maximum weight indicated. Users must be taller than the minimum height indicated.

ASSEMBLY AND INSTALLATION

Assemble the screws and cover (as applicable) as shown in figure 1. Position the seat over the toilet, adjust the screws and secure the seat before using.

PRECAUTIONS

The construction material is inflammable. Do not expose the products to situations that could set them on fire. In the event of a fire, quickly let go of them and use the

proper resources to put it out. Do not expose the product to direct heat sources such as stoves, heaters, radiators, etc.

SAFETY AND MAINTENANCE

· Storage: keep within a temperature range of 0-40°C in a cool, dry place.

· Cleaning: clean with a wet sponge using warm water and detergent. Rinse with cold water and dry with a soft, dry cloth. Check the status of the screws for extended use.

Remove any dirt with a damp cloth and neutral soap, as necessary. Then dry. Do not use any aggressive or abrasive chemical cleaners. They may damage the product material.

If contaminated with infected material, use standard disinfection resources.

WARRANTY

ORLIMAN, S.L.U. guarantees all its products as long as the original configuration has not been manipulated or altered. It does not guarantee any products with altered characteristics due to improper use, defects or breakage of any kind. The product is covered by a two-year warranty for material or manufacturing defects. Excluded from the warranty are imperfections caused by improper use, accidents or natural wear of parts such as plastic components, footrests, upholstery, brakes, tips, inner tubes and wheelchair tyres and similar parts, depending on the type of medical product acquired.

If you do not follow the instructions, warnings and the notes in the user's manual, you or a third party may be physically injured or the product may be damaged.

If you observe any defect, anomaly or failure, notify the dealer immediately so it may be replaced or repaired.

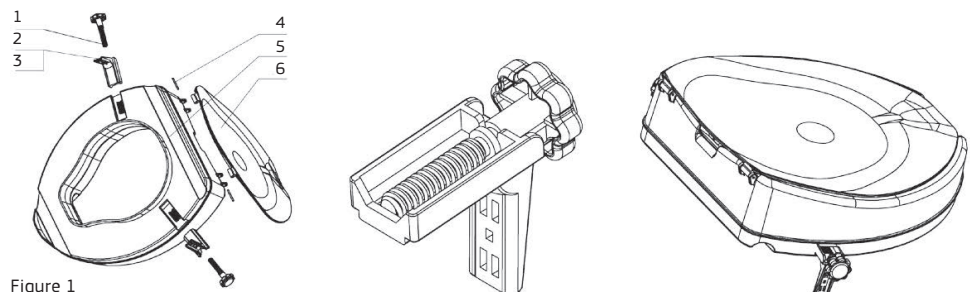
Figure 1

	ACAB04#UNI	ACAB05#UNI	ACAB06#UNI	ACAB07#UNI
Height	100 mm	100 mm	150 mm	150 mm
Width	410 mm	410 mm	410 mm	410 mm
Maximum occupant weight	100 kg	100 kg	150 kg	150 kg
Recommended for people who are at least	1.55 m	1.55 m	1.70 m	1.70 m

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