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ACAB11#UNI

Dear Customer,

Thank you very much for your trust in an Actius by Orliman product. You have purchased a quality product. Please read the instructions carefully before using the product and use the product only as described and for the intended purposes.

If you have any questions about the use, maintenance or safety of your product, contact the healthcare establishment where you purchased the product or our customer service department. Keep these instructions for future reference.

REGULATIONS

This product is compliant with the European Directive on Medical Devices 93/42/EEC (Spanish Royal Decree 1591/2009). A Risk Analysis (EN ISO 14971) was done, minimising all existing risks.

Strictly observe municipal ordinance when discarding the packaging and product.

SUITABLE FOR

Recommended for people with little balance or a motor disability; they provide patient safety and stability when showering or when being washed. The occupant must not weigh more than the maximum weight indicated.

ASSEMBLY AND INSTALLATION

1. Screw the legs into the bench or chair.
- (2) Adjust the adjustment tubes to the desired height, as indicated in the diagram, until you hear the lock pins click when they're perfectly inside their housings.
3. Make sure the 4 legs are level and at the same height.
4. Make sure the legs are firmly in contact with the ground, bathtub or shower before each use.
5. Stow the back pad as indicated in the diagram for the chair model.

PRECAUTIONS

The construction material is inflammable. Do not expose the products to situations that could set them on fire. In the event of a fire, quickly let go of them and use the proper resources to put it out. Do not expose the product to direct heat sources such as stoves, heaters, radiators, etc.

RECOMMENDATIONS AND WARNINGS

- Carefully read these instructions before the user uses the sill or bench.

- The bath chair or bench is for sitting down in the shower or bathtub. It must only be used over showers or bathtubs with stable, non-skid floors. This product is designed for private use only and is not suitable for hospital use.

- This technical assistance must not be used for any other purpose except as intended. Do not use this product as a step stool. Do not stand on it. Do not use on furnishings. Do not allow children to play with this device.

- Do not modify the product or add other elements.

- Inspect periodically and before using to ensure the fixing elements are firmly secure. Carefully check the conditions of all components. If any are not in perfect conditions of use, they must be replaced with an original spare part.

- The skin in contact with pressure support areas must not be wounded or hyper-sensitive. Check for any redness on the skin, pressure or patient discomfort. Do not use in cases of irritation, pressure, pain or obvious discomfort.

SAFETY AND MAINTENANCE

- Clean and dry the chair after each use. Remove any dirt with a damp cloth and neutral soap, as necessary. Then dry. Do not use any aggressive or abrasive chemical cleaners. They may damage the product material.

- If contaminated with infected material, use standard disinfection resources.

WARRANTY

ORLIMAN, S.L.U. guarantees all its products as long as the original configuration has not been manipulated or altered. It does not guarantee any products with altered characteristics due to improper use, defects or breakage of any kind. The product is covered by a two-year warranty for material or manufacturing defects. Excluded from the warranty are imperfections caused by improper use, accidents or natural wear of parts such as plastic components, footrests, upholstery, brakes, tips, inner tubes and wheelchair tyres and similar parts, depending on the type of medical product acquired.

If you do not follow the instructions, warnings and the notes in the user's manual, you or a third party may be physically injured or the product may be damaged.

If you observe any defect, anomaly or failure, notify the dealer immediately so it may be replaced or repaired.

	ACAB08#UNI	ACAB09#UNI	ACAB10#UNI	ACAB11#UNI
Depth	390 mm	480 mm	500 mm	420 mm
Width	520 mm	500 mm	480 mm	530 mm
Min. - max. seat height	420-520 mm	360-510 mm	420-520 mm	420-520 mm
Maximum occupant weight	100 kg	100 kg	100 kg	100 kg
Seat depth	300 mm	300 mm	380 mm	380 mm
Seat width	400 mm	400 mm	420 mm	420 mm
Total height including the back pad	-	710-860 mm	-	-

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