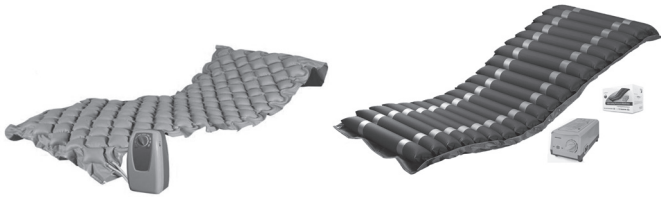
## ▶ COLCHÓN DE AIRE ANTIESCARAS

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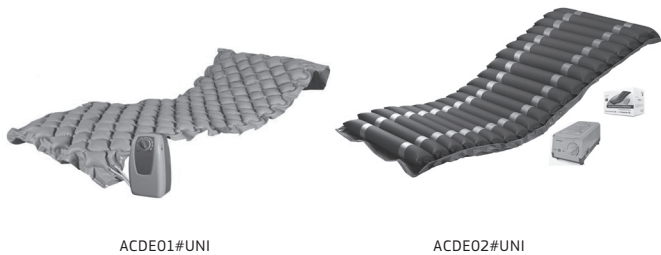
Dear Customer,

Thank you very much for your trust in an Actius by Orliman product. You have purchased a quality product. Please read the instructions carefully before using the product and use the product only as described and for the intended purposes.

If you have any questions about the use, maintenance or safety of your product, contact the healthcare establishment where you purchased the product or our customer service department. Keep these instructions for future reference. Read these instructions before using the product.

#### REGULATIONS

This product is compliant with the European Directive on Medical Devices 93/42/EEC (Spanish Royal Decree 1591/2009). A Risk Analysis (EN ISO 14971) was done, minimising all existing risks.

Strictly observe municipal ordinance when discarding the packaging and product.

#### SUITABLE FOR

This mattress is recommended to prevent pressures ulcers up to Grade I (ACDE01#UNI), Grade II (ACDE02#UNI) and as an auxiliary method to treat patients who spend long periods of time in bed. It is used as a mattress cover; therefore a base mattress is required for proper use. The user must not weigh more than the maximum weight indicated.

#### COUNTERINDICATIONS

Patient conditions for which therapy using an alternating pressure system is not recommended:

- Cervical or skeletal traction.
- Unstable marrow dysfunction.

#### PRODUCT DESCRIPTION:

Pump

1. Switch: The switch is on the right side of the pump. By turning the switch ON/OFF, the Start/Stop pump begins/stops operating. 2. Air valves: Connect the air tubes to the air valves. 3. Pressure adjustment button: Turn the Min./Max. button to a comfortable pressure level.

Characteristics

- Output ACDE01 5-8 L/min, ACDE02 6-7 L/min
- Adjustable pressure button.
- 6-minute cycle.
- Built-in bed assembly supports.
- 2m power cable.

Alternating pressure mattress

- Dimensions: 200cm Long x 90cm Wide x 7cm High
- 130 individual bubble cells for max. surface area
- Straps to position and secure the alternating pressure mattress.
- Maximum user weight: ACDE01 120 kg, ACDE02 145 kg

#### USE

Step 1. Position the alternating pressure mattress. The air tubes must be at the foot of the bed where the air valves on the left side of the pump can be easily connected.

Step 2. Cover it with a cotton sheet to prevent direct contact with the skin and reduce friction.

Step 3. Hang the pump over the frame at the foot of the bed. Make sure the pump is secure.

Step 4. Connect the alternating pressure mattress air tubes to the pump air valves. Make sure they are correctly connected.

ATTENTION: Make sure the air tubes are not bent or under the alternating pressure mattress.

Step 5. Plug in the power cable

NOTE: Before plugging in, make sure the voltage is compatible.

Step 6. Connect the power by pressing the on/off switch to the right of the pump. See the operating section.

Step 7. Make sure the pump is turned off and the power cable is unplugged when not in use.

#### STEP-BY-STEP OPERATION:

Pump APP

Step 1. Turn it on. The switch light will turn on. The pump will begin pumping air into the pillow.

Step 2. The first time it is used, turn the pressure button all the way up.

Step 3. After fully inflating it, turn the pressure button and adjust the pressure from gentle to strong in accordance with the patient's weight and preferences.

Set the pressure. The pressure button should be at the maximum setting when the pillow is blown up for the first time. Users can then easily adjust the pillow pressure level to the desired firmness or as recommended by a professional.

If the pressure is low, check for any leaks in the air tubes. Contact your local distributor to replace damaged tubes.

#### CLEANING:

Pump

Do not submerge or soak the pump in water. Check for any external damage and move the pump to a cleaning area. Place the pump on a work surface and clean the outside of the box with a quaternary ammonium solution. Do not use any cleaning product directly on the pump surface. Do not use any hypochlorite or phenol-based cleaning product as it could damage the box.

The solution should remain on the product for 10 min. or as indicated on the cleaning product used. Clean the front with a cloth. Do NOT allow any excess cleaning product on the control panel or front of the box. If the product gets inside the box, it will damage the pump. Let the surface completely dry after cleaning.

When the pump is completely clean and dry, plug in the pump and test it to guarantee normal operation.

Alternating pressure mattress

Any blood stain must be moistened with a 1:9 hypochlorite solution (1 part bleach and 9 parts water). Let dry for at least 10 min. Then dry with a clean, damp cloth. Let air dry completely.

Once dry inside, turn it over: clean the outside of the alternating pressure mattress.

#### PRECAUTIONS

The construction material is inflammable. Do not expose the products to situations that could set them on fire. In the event of a fire, quickly let go of them and use the proper resources to put it out. Do not expose the product to direct heat sources such as stoves, heaters, radiators, etc.

#### RECOMMENDATIONS AND WARNINGS

Follow these safety measures when electrical products are used and particularly in the presence of children:

**DANGER** ⚠ (to reduce the risk of electrical shock).

- Unplug immediately after use.
- Do not use while bathing.
- Do not place or store the product where it may fall into water.
- Do not place or let fall in water or any other liquid.
- Do not touch the product after it has fallen into water. Immediately unplug.

**WARNINGS** ⚠ (to reduce the risk of burns, electrical shock, fire or injury).

- The product may not be left alone when plugged in.
- The product must be supervised by an adult and kept out of the reach of children.
- Use this product only for the intended use as described in this manual. Do not use accessories not recommended by the manufacturer.
- A broken cable or socket. If it does not work properly or has fallen, been broken or submerged in water, call an authorised distributor to repair or replace it.
- Keep the power cable far from hot surfaces.
- Do not cover air outlets on the product or leave on any type of soft surface such as a bed or sofa where the air outlets may get blocked.
- Keep the air outlets free of fuzz, hair and dust.
- Do not let it fall or insert any object into any opening or the hose.
- Do not use outside or where aerosols (sprays) or oxygen are used.
- Connect this product only to earthed sockets. See the instructions.

#### STORAGE AND MAINTENANCE

##### STORAGE

Smooth out the alternating pressure mattress. Carefully fold or roll. Unplug the pump and store with the identification label. Follow national regulations when disposing of the pump.

##### MAINTENANCE

Periodically check the power cable and socket for any abrasions or excessive wear. Plug in the pump and check the air flow from the hose connection port. The air flow must alternate between ports every half-cycle.

Keep in a dry place at room temperature.

#### WARRANTY

ORLIMAN, S.L.U. guarantees all its products as long as the original configuration has not been manipulated or altered. It does not guarantee any products with altered characteristics due to improper use, defects or breakage of any kind. The compressor comes with a 2-year warranty and the mattress, a 6-month warranty, for material or manufacturing defects. Excluded from the warranty are imperfections caused by improper use, accidents or natural wear of parts such as plastic components, footrests, upholstery, brakes, tips, inner tubes and wheelchair tyres and similar parts, depending on the type of medical product acquired.

If you do not follow the instructions, warnings and the notes in the user's manual, you or a third party may be physically injured or the product may be damaged.

If you observe any defect, anomaly or failure, notify the dealer immediately so it may be replaced or repaired.

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