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Thank you for buying a high quality Actius by Orliman® product. Please read the following instructions and retain them for future reference. Use the product only as described, and for its intended purpose.

**SUITABLE FOR****ACE302**

- Shoulder dislocation. Postoperative recovery.

**ACE402**

- Epicondylitis (tennis elbow). Medial Epicondylitis (golfer's elbow).

**ACE813**

- Osgood Schlatter. Patellar tendonitis.

**PLACEMENT**

In order to ensure a correct product fit, it is essential to choose the correct size (the packaging shows a size guidance chart with centimetre equivalents. Be aware that the garment may come in a universal size).

To fit the product, place it on the corresponding joint segment, ensuring it is aligned or adjusted to the muscle movement. Adjust the product to ensure it permits natural movement and does not cause any irritation or discomfort when used.

Follow the pictures (if given). We advise that you tighten the product sufficiently to achieve a snug fit, without applying excessive pressure.

**ACE302**

- 1-Place the arm inside the bag and insert the thumb through the inner passage.
- 2-Pull the suspension strap over the opposite shoulder.
- 3-Adjust the closure as well as the height of the arm and elbow flex.

**ACE402**

- 1-Wrap the strap around the forearm (under the elbow) so the pad is in the painful area.
- 2-Close by pulling the adjustment strap through the clasp.
- 3-Adjust the pressure and close the strap with the quick closure.

**ACE813**

- 1-Wrap the strap around the leg so the silicone patella support is just below the kneecap.
- 2-Tighten the fastening straps equally to the desired pressure, making sure the patella tendon support is centred.
- 3-Secure both straps using the quick closure.

**RECOMMENDED CARE INSTRUCTIONS**

Hand wash from time to time using warm water, pH neutral soap, and allow to dry naturally. Store in a dry place at room temperature.

**WARNING**

Do not use this product on an open wound. Should irritation occur, stop using the product and inform your doctor. Do not place near heat sources and do not combine with other products such as creams or ointments.

**KEEP OUT OF THE REACH OF CHILDREN.**

This product is a class I healthcare product according to the Medical Devices Directive 93/42/EEC.

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