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Thank you for buying a high quality Actius by Orliman® product. Please read the following instructions and retain them for future reference. Use the product only as described, and for its intended purpose.

#### SUITABLE FOR

ACE601

- Back pain during pregnancy.

ACE605 / ACE606 / ACE607 / ACE608 / ACE609

- Back pain, sciatica pain, post-operative support.

#### PLACEMENT

In order to ensure a correct product fit, it is essential to choose the correct size (the packaging shows a size guidance chart with centimetre equivalents). Be aware that the garment may come in a universal size).

To fit the product, place it on the corresponding joint segment, ensuring it is aligned or adjusted to the muscle movement. Adjust the product to ensure it permits natural movement and does not cause any irritation or discomfort when used.

Follow the pictures (if given). We advise that you tighten the product sufficiently to achieve a snug fit, without applying excessive pressure.

- 1-Position the back support around the lower back so it is centred with the lumbosacral segment.
- 2-Stretch both sides of the back support and then secure at the abdomen with the quick closure.
- 3-Tighten the ends of the tensing strips towards the abdomen.
- 4-Secure the tightening strips using the quick-close straps above the previously adjusted back support.

#### RECOMMENDED CARE INSTRUCTIONS

Hand wash from time to time using warm water, pH neutral soap, and allow to dry naturally. Store in a dry place at room temperature.

#### WARNING

Do not use this product on an open wound. Should irritation occur, stop using the product and inform your doctor. Do not place near heat sources and do not combine with other products such as creams or ointments.

#### KEEP OUT OF THE REACH OF CHILDREN.

This product is a class I healthcare product according to the Medical Devices Directive 93/42/EEC.

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