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Thank you for buying a high quality Actius by Orliman® product. Please read the following instructions and retain them for future reference. Use the product only as described, and for its intended purpose.

SUITABLE FOR

ACE614 / ACE615 / ACE616

- To hold minor hernias.
- Postoperative recovery.

ACE618 / ACV601

- Backache.
- Posture correction.

ACE617

- After ostomy surgery.

PLACEMENT

In order to ensure a correct product fit, it is essential to choose the correct size (the packaging shows a size guidance chart with centimetre equivalents. Be aware that the garment may come in a universal size). To fit the product, place it on the corresponding joint segment, ensuring it is aligned or adjusted to the muscle movement. Adjust the product to ensure it permits natural movement and does not cause any irritation or discomfort when used.

Follow the pictures (if given). We advise that you tighten the product sufficiently to achieve a snug fit, without applying excessive pressure.

ACE614 / ACE615 / ACE616

- 1-Wrap the band around the affected area.
- 2-Close the quick-close areas.

ACE618 / ACV601

- 1-Insert the arms into the holes.
- 2-Tighten the straps in the front until the back is in a correct and comfortable position.

ACE617

- 1-Then place the back support around the back.
- 2-Mark the centre of the stoma.
- 3-Depending on the diameter of the bag used by the patient, draw a circle and carefully cut a hole in the fabric and in the reinforcement plate.
- 4-Insert the bag through the hole and adjust the garment.
- 5-Adjust the band above the bag.

RECOMMENDED CARE INSTRUCTIONS

Hand wash from time to time using warm water, pH neutral soap, and allow to dry naturally. Store in a dry place at room temperature.

**WARNING**

Do not use this product on an open wound. Should irritation occur, stop using the product and inform your doctor. Do not place near heat sources and do not combine with other products such as creams or ointments.

KEEP OUT OF THE REACH OF CHILDREN.

This product is a class I healthcare product according to the Medical Devices Directive 93/42/EEC.

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