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Thank you for buying a high quality Actius by Orliman® product. Please read the following instructions and retain them for future reference. Use the product only as described, and for its intended purpose.

**SUITABLE FOR****ACE806**

- Grade 2 or 3 sprains. Gonarthrosis. Chronic knee instability. Post-traumatic or postoperative recovery.

**ACE810**

- Cruciate, collateral or patellar ligament rupture and instability. Sprain. Patellofemoral pain. Arthrosis. Post-traumatic or postoperative recovery.

**ACE811**

- Stabilization during sports activities or stress situations. Patients requiring quick and easy placement due to special morphology.

**PLACEMENT**

In order to ensure a correct product fit, it is essential to choose the correct size (the packaging shows a size guidance chart with centimetre equivalents. Be aware that the garment may come in a universal size).

To fit the product, place it on the corresponding joint segment, ensuring it is aligned or adjusted to the muscle movement. Adjust the product to ensure it permits natural movement and does not cause any irritation or discomfort when used.

Follow the pictures (if given). We advise that you tighten the product sufficiently to achieve a snug fit, without applying excessive pressure.

**ACE806**

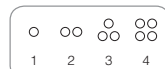
- 1-Set the allowed joint movement range as per a medical prescription. To do so, lift the cover over the joint and put the lock pins into the desired positions. Close the protective cover.
- 2-Insert the knee brace so the semi-circular patellar donut surrounds the lower edge of the kneecap.
- 3-Close the thigh band with the quick closure.
- 4-Adjust the pressure with the top and bottom straps and then close by inserting them through the clasps and fastening with the quick closure.
- 5-Make sure the joints are parallel and aligned with the medial and lateral axis of the knee.

**ACE810**

**BASED ON A MEDICAL PRESCRIPTION OR PATHOLOGY:** If necessary, replace the metal articulations with spiral stabilisers.

- 1-Insert the knee brace so the patellar donut surrounds the outer edges of the kneecap and the knee brace is centred.
- 2-Place the hand in the finger inserts at the back of the thigh.
- 3-Adjust the desired pressure on the thigh and calf by closing the quick-closure straps.

If necessary, put the strap in an "X" position to reinforce the ligaments. There are 2 possible positions for this strap. The ends are marked 1 to 4 (check the diagrams):

**A-POSITIONING FOR COLLATERAL LIGAMENTS**

- 1-Position the strap at back of the knee.
- 2-Adjust wings 1 and 2, wrapping the knee and securing with the quick closure.
- 3-Finish by folding wings 3 and 4 around the leg and securing with the quick closure.

**B-POSITIONING FOR CRUCIATE LIGAMENTS**

- 1-Position the strap at front of the knee.
- 2-Adjust wings 1 and 2, wrapping the knee and securing with the quick closure.
- 3-Finish by folding wings 3 and 4 around the leg and securing with the quick closure.

**ACE811**

- 1-Wrap the knee brace so the patellar donut surrounds the outer edges of the kneecap.
- 2-Close the top and bottom straps with the quick closure.
- 3-The side strips must be parallel and aligned with the medial and lateral axis of the knee.

**RECOMMENDED CARE INSTRUCTIONS**

Hand wash from time to time using warm water, pH neutral soap, and allow to dry naturally. Store in a dry place at room temperature.

**WARNING**

Do not use this product on an open wound. Should irritation occur, stop using the product and inform your doctor. Do not place near heat sources and do not combine with other products such as creams or ointments.

**KEEP OUT OF THE REACH OF CHILDREN.**

This product is a class I healthcare product according to the Medical Devices Directive 93/42/EEC.

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