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Thank you for buying a high quality Actius by Orliman® product. Please read the following instructions and retain them for future reference. Use the product only as described, and for its intended purpose.

SUITABLE FOR

- Sprained ankle.
- Postoperative or post-traumatic recovery.

PLACEMENT

In order to ensure a correct product fit, it is essential to choose the correct size (the packaging shows a size guidance chart with centimetre equivalents. Be aware that the garment may come in a universal size).

To fit the product, place it on the corresponding joint segment, ensuring it is aligned or adjusted to the muscle movement. Adjust the product to ensure it permits natural movement and does not cause any irritation or discomfort when used.

Follow the pictures (if given). We advise that you tighten the product sufficiently to achieve a snug fit, without applying excessive pressure.

ACE903

- 1-Release the top laces as much as possible to easily insert the foot, correctly positioning the heel and avoiding any tongue bending.
- 2-Secure the quick-close fastener by tightening the instep laces. Then, adjust the top laces.
- 3-Tightening the lateral stabilising strap (outside) first is recommended. To do so, cross it over the instep and place it under the foot to secure it vertically. The colours help identify the fixed position.
- 4-Repeat with the medial strap (inside).
- 5-Wrap the ankle with the top elastic strap.

ACE904

- 1-Insert the ankle in the ankle brace, carefully positioning the ankle in the correct position.
- 2-Wrap the ankle and tighten the elastic strap. Close the end of the strap over it.

ACV901

- 1-Open the ankle brace and place the foot in the middle of the calcaneus heel bone strap.
- 2-While holding the plates, position the quick-close strips and adjust the straps beginning with the inner one.
- 3-Adjust the calcaneus heel bone strap by pulling it upwards and connecting both sides to the quick-close strips.

ACV902

The product is comprised of 3 components and may be used in 3 ways:

Conventional ankle brace:

Without any accessories, the ankle brace works as a protective ankle brace.

Open all the straps and place the foot in the brace with the heel in the correct opening and close the quick-close straps.

Using the malleolus plates:

Adapting the rigid plates offers more medial or lateral stabilisation or on both sides. The plates must be applied before putting the ankle brace on.

Depending on the ankle affected, insert the "INTERIOR-INSIDE" and "EXTERIOR-OUTSIDE" plates in the corresponding pockets. It's possible to use just one plate, but the stabilising effect is more effective when both are used.

Using the corrective strap:

Adapting the tightening strap provides more protection for inversion and eversion movements.

- 1-The strap must be applied last.
- 2-The end of the rough closure after the metal clasp must be placed at the bottom of the midfoot and diagonally depending on the side to be protected (outer or inner).
- 3-Wrap the ankle, tighten the strap and close it horizontally on the top of the ankle.
- 4-After positioning the 2 rough closures, reinforce the strap fit by unfastening and tightening the end area just before the clasp.

ACV903

Placing the walker in a sitting position and putting a sock (cotton, if possible) on the patient's leg and foot is recommended.

- 1-Unfasten the quick closures to separate the rigid structure from the wrap-around sleeve. Then, place the wrap-around sleeve snug against the patient's foot and leg and tighten it as much as possible using the quick-close system. Positioning the protective toe pad.
 - 2-Insert the leg with the sleeve into the rigid structure. Make sure the foot is in direct contact with the base. Make sure the mid-lateral strips are parallel to each other and centred on the leg and ankle load line. Secure the mid-lateral strips to the wrap-around sleeve using the quick closures on the inside.
 - 3-Place the protective tibial and calf plates in the straps and close and secure these quick-close straps.
- Adjusting the articulation
- For proper articulation, do not graduate the two extension and flexing movements at the same time.
- 1-Unlock the white adjuster by sliding the orange part.
 - 2-Press the white part while moving to desired degrees. Make sure you hear a click.
 - 3-Slide the orange part again all the way down the white part to lock the position. This prevents it from moving or changing the chosen degrees.
 - 4-Do the same on both sides of the joint.
- (To lock the rods in a fixed 90° angle, position the flexing and extension adjustments at 0°).

RECOMMENDED CARE INSTRUCTIONS

Hand wash from time to time using warm water, pH neutral soap, and allow to dry naturally. Store in a dry place at room temperature.

WARNING

Do not use this product on an open wound. Should irritation occur, stop using the product and inform your doctor. Do not place near heat sources and do not combine with other products such as creams or ointments.

KEEP OUT OF THE REACH OF CHILDREN.

This product is a class I healthcare product according to the Medical Devices Directive 93/42/EEC.

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