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Thank you for buying a high quality Actius by Orliman® product. Please read the following instructions and retain them for future reference. Use the product only as described, and for its intended purpose.

#### SUITABLE FOR

ACN401 / ACN402 / ACN501 / ACN502 / ACN601 / ACN602 / ACN801 / ACN802 / ACN901

- Prevention of or recovery from muscle and/or joint injuries.
- Thermo-pressure (heat effect) for sports, work or other daily activities.

#### ACN503

- Tendinitis. Rhizarthrosis. Sprain. Instability. Postoperative or post-traumatic recovery.

#### PLACEMENT

In order to ensure a correct product fit, it is essential to choose the correct size (the packaging shows a size guidance chart with centimetre equivalents). Be aware that the garment may come in a universal size).

To fit the product, place it on the corresponding joint segment, ensuring it is aligned or adjusted to the muscle movement. Adjust the product to ensure it permits natural movement and does not cause any irritation or discomfort when used.

Follow the pictures (if given). We advise that you tighten the product sufficiently to achieve a snug fit, without applying excessive pressure.

ACN401 / ACN402 / ACN501 / ACN502 / ACN601 / ACN602 / ACN801 / ACN802 / ACN901

1-Place the product on the corresponding joint segment (trunk, elbow, hand, arm, knee or ankle), ensuring it is aligned or adjusted to the muscle movement.

2-Adjust the product with the quick-close straps for firm yet comfortable pressure.

#### ACN503

1-The product is comprised of two adjustable splints - one for the palm and another for the thumb.

2-Before putting the wrist brace on, place palm splint 2 in the sleeve for the affected thumb:

- The sleeve with 1 mark for the right thumb.
- The sleeve with 2 marks for the left thumb.

3-Put the wrist brace on by inserting the thumb through the opening for this purpose.

4-Wrap the double strap around the wrist, adjusting it to the desired pressure with the inner quick closure.

5-Tighten the wide section of the double strap, adjust to the desired pressure and then secure with the quick closure. Do the same with the thin section of the double strap.

6-If necessary, mould the circular part of the thumb splint for the digit shape. Adjust the thumb strap and secure with the quick closure.

#### RECOMMENDED CARE INSTRUCTIONS

Hand wash from time to time using warm water, pH neutral soap, and allow to dry naturally. Store in a dry place at room temperature.

#### WARNING

Do not use this product on an open wound. Should irritation occur, stop using the product and inform your doctor. Do not place near heat sources and do not combine with other products such as creams or ointments.

#### KEEP OUT OF THE REACH OF CHILDREN.

This product is a class I healthcare product according to the Medical Devices Directive 93/42/EEC.

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