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Thank you for buying a high quality Actius by Orliman® product. Please read the following instructions and retain them for future reference. Use the product only as described, and for its intended purpose.

SUITABLE FOR

ACV501

- Pulled extensor on the dorsal surface of the distal phalanx. Other pain due to direct trauma in the distal phalanx.

ACV502 + ACV503+ ACV504

- Pain caused by direct trauma to the fingers.

PLACEMENT

In order to ensure a correct product fit, it is essential to choose the correct size (the packaging shows a size guidance chart with centimetre equivalents). Be aware that the garment may come in a universal size). To fit the product, place it on the corresponding joint segment, ensuring it is aligned or adjusted to the muscle movement. Adjust the product to ensure it permits natural movement and does not cause any irritation or discomfort when used.

Follow the pictures (if given). We advise that you tighten the product sufficiently to achieve a snug fit, without applying excessive pressure.

ACV501

- 1- Insert the finger in the splint.
- 2-Secure with adhesive tape around the upper splint tab behind the distal interphalangeal joint to properly position the finger and stabilise the splint.

ACV502

- 1-Insert the finger in the splint, adjusting it to the desired position and proper immobilisation.
- 2-Wrap the finger to secure it using the wings.

ACV503+ACV504

- 1-Insert the finger in the splint.
- 2-Adjust the aluminium splints to the desired finger position and proper immobilisation.

RECOMMENDED CARE INSTRUCTIONS

Hand wash from time to time using warm water, pH neutral soap, and allow to dry naturally. Store in a dry place at room temperature.

WARNING

Do not use this product on an open wound. Should irritation occur, stop using the product and inform your doctor. Do not place near heat sources and do not combine with other products such as creams or ointments.

KEEP OUT OF THE REACH OF CHILDREN.

This product is a class I healthcare product according to the Medical Devices Directive 93/42/EEC.

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