ES

EN

actius[®]
by  **ORLIMAN**[®]

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2. INDICATIONS

3. PRODUCT IDENTIFICATION

4. GENERAL DIAGRAM OF YOUR WHEELCHAIR

4.1. Wheelchair components

4.2. General explanation of the operation and options for your wheelchair

5. SAFETY INSTRUCTIONS

5.1. General safety rules

5.2. Warnings for safe use

6. INSTRUCTIONS FOR USE

6.1. Safety tips

6.2. Using on uneven surfaces

6.3. Going up and down ramps

6.4. Going up and down stairs

7. INSTRUCTIONS FOR USE AND MOUNTING OPTIONS

7.1. Technical information

7.2. Unfolding and folding the wheelchair

7.3. Using the footrest

7.4. Using the brakes

7.5. Using the armrest

7.6. Using the rear wheel

7.7. Table with wheel models

7.8. Using the fold-down backrest

7.9. Using the accessories and other wheelchair options

8. TRANSPORT AND TRANSIT BY CAR

8.1. De-assembly

8.2. Assembly

9. MAINTENANCE

9.1. Warranty

9.2. Wheels

9.3. Brakes

9.4. Cross frames

9.5. Rear wheels

9.6. Front wheels and front castor forks

9.7. Upholstery

9.8. Cleaning

10. TROUBLESHOOTING

Dear Customer,

Thank you very much for your trust in an Actius by Orliman product. You have purchased a quality product. Please read the instructions carefully before using the product and use the product only as described and for the intended purposes.

If you have any questions about the use, maintenance or safety of your product, contact the healthcare establishment where you purchased the product or our customer service department.

1. REGULATIONS

This product is compliant with the European Directive on Medical Devices 93/42/EEC (Spanish Royal Decree 1591/2009). A Risk Analysis (EN ISO 14971) was done, minimising all existing risks.

Strictly observe municipal ordinance when discarding the packaging and product.

2. INDICATIONS

Increase mobility with self-propelled movement or an assistant for people who can't walk or with reduced mobility inside and outside as long as the ground is firm, level and dry. The occupant must not weigh more than the maximum weight indicated.

3. PRODUCT IDENTIFICATION

Your wheelchair is equipped with a unique serial number. You can find this number on the frame of your wheelchair. Below is an example of the frame label where you can find the identification number. You will also find an explanation of the different types of information included on the label, as indicated below.

1. Model name: the name of the model of your wheelchair.
2. Serial number: each wheelchair has its own unique serial number. You must have this number when making technical queries. The serial number will also be required for parts under warranty.
3. Maximum occupant weight: the highest occupant weight allowed for wheelchair and occupant safety.
4. Production date: the date your wheelchair was manufactured.
5. Model number: this number indicates the wheelchair model you have and will also always be required for technical queries.

4. GENERAL DIAGRAM OF YOUR WHEELCHAIR

4.1. Wheelchair components

1	Backrest pad upholstery
2	Height adjustable armrest
3	Upholstered seat
4	Footrest
5	Cross frame
6	Heel strap
7	Footplate
8	Front wheels
9	Front castor
10	Bearing
11	Stepper
12	Brakes
13	Hand rim
14	Rear wheels
15	Lock lever for foldable backrest
16	Push handle/grips

Your wheelchair is equipped with various elements and parts. You should know where they are before you continue reading this manual. The design and specifications may change without prior notice.

4.2. General explanation of the operation and options for your wheelchair

Your wheelchair is foldable and designed so that it is easy to transport. Your wheelchair is a medical device and is not a standard consumer product.

You must follow this manual completely to ensure optimal and safe use of your wheelchair.

5. SAFETY INSTRUCTIONS

Under no circumstance will ORLIMAN be liable for any bodily injury or property damage which may occur when using the wheelchair contrary to the laws or ordinances. If used correctly, the product offers the utmost safety and stability as long as the instructions for use described in this manual are followed. However, dangerous situations may occur when the wheelchair is not used correctly.

5.1. General safety rules

Protect your wheelchair by checking it regularly. Dangerous situations may occur when any part on the wheelchair is not working properly.

ⓘ **Warning:** You must keep your wheelchair in good conditions to guarantee safe use.

5.2. Warnings for safe use

ⓘ Warnings:

- Please carefully read the instructions for use before using the product. You must be familiar with how to handle it and how it works before using. Your safety may be jeopardised if you do not pay attention to these instructions. Also remember that complete protection is impossible as is preventing unexpected circumstances and situations.
- Do not go up or down stairs without the help of an assistant. You should use ramps or lifts if available. If there are no available infrastructures for wheelchairs, the chair must be maneuvered by two people to overcome any obstacles.
- Accidents may occur when maneuvering the wheelchair without experience. Falls and other dangerous situations may occur. Therefore, make yourself familiar with the new wheelchair by practising on firm, smooth ground. Learn how the wheelchair works upon changes in the centre of gravity on ramps or sloped ground or when getting around obstacles like stairs or curves, for example. This must be done with the help of another person.
- Not following the instructions described in this manual may cause damage to the wheelchair or bodily injury to the occupant.
- Avoid using your wheelchair on roads. Use pavements if possible.
- Do not use your wheelchair in sand or on irregular, wet or slippery surfaces or when there is a lack of adherence.
- Do not use the wheelchair when it is broken or is not working properly.
- These standard wheelchairs are not approved for transport in handicapped adapted vehicles. Wheelchair users must always be transferred to adapted seats in a vehicle. If these instructions are not observed, the wheelchair could fail in the event of an accident and cause harm to the occupant as well as any other passenger in the vehicle.
- Do not recline backwards when in the wheelchair. This can cause the wheelchair to tip over.
- Before leaning forward or beginning to move, sit firmly back in the chair and turn the wheels forwards in parallel to each other.
- Do not push the wheelchair near kerbs.
- Before leaning forwards sit firmly against the backrest and make sure that the front wheels are facing forward.

- Never connect anything to the wheels. This may damage the wheelchair and negatively affect the wheelchair balance and harm the occupant.
- Engage the brakes when you want to get up or someone is lifting you up. The brakes on the wheels do not brake the forward movement but are rather designed to lock the wheel when the chair is stopped. The wheel brakes must not be used as a way to slow down because suddenly locking the wheelchair could cause falls in extreme situations.
- The maximum weight capacity is indicated on your wheelchair frame label and in the TECHNICAL DATA section. The maximum occupant weight includes the occupant's weight plus the weight of the accessories installed on the wheelchair and any other built-in element (shopping bag, portable oxygen equipment, etc.).
- For suitable protection from potential obstruction, the footrests should be a minimum of 7 cm off the ground.

Figure 1

- Do not stand on the footrests as this could cause the wheelchair to tip over and injure the occupant. (Figure 1)
- Position the wheelchair on a stable, flat surface and engage the brakes before transferring to or from the wheelchair.
- Unauthorised modifications and/or use of parts not supplied by ORLIMAN will invalidate the warranty on this wheelchair and may lead to occupant injury and/or damage to the wheelchair.

6. INSTRUCTIONS FOR USE

You should learn all about the characteristics of your wheelchair. It is of utmost importance to understand the safety measures in order to do all daily activities in accordance with your lifestyle. Ask your doctor or therapist for assistance in order to learn the most effective techniques for your daily activities in accordance with your physical possibilities.

6.1. Safety tips

Daily tasks such as getting out of the wheelchair, bending or reaching for something from the wheelchair will cause a change of weight distribution and the centre of gravity between you and your wheelchair. You will need to perform these movements and techniques as described below.

→ Transferring:

Transferring in a wheelchair is a difficult manoeuvre. Ask your physical therapist for assistance in developing your individual technique. Your transfer possibilities depend on your physical condition. We advise you to discuss this with your physical therapist first.

However, if you have sufficient body strength and you want to transfer to and from the wheelchair, the best way to do so is as follows:

→ To get out of the wheelchair:

First of all, you must make sure that your wheelchair is as close as possible to the place you wish to move to. Position the wheels forward and engage the brakes. Swing away the footplates and place your feet on the ground. You are now in a position to carry out the task. Shift your weight to the front of the chair, using the armrests for support if you wish, and move yourself to the place you wish to move to. You could use a transfer board, if necessary.

→ To get in:

You need to take the same steps to get in as when you want to get out of the wheelchair, only in reverse order.

→ Reaching from the wheelchair:

We strongly recommend that you seek the help of an assistant or caretaker if you wish to pick up something off the ground.

6.2. Using on uneven surfaces

→ Getting up onto pavement:

Your caretaker or assistant can help you in two different ways; however, we advise you to do it the first way.

1. The first method for getting up onto pavement involves you and your assistant facing forward when doing so. Your assistant must push the grips towards their body and put one foot on the stepper (see figure 2.1). A leverage effect occurs that inclines the wheelchair when the assistant does the two tasks simultaneously, indicated in figure 2.

Figure 2

2. With the second method, the assistant first gets on the pavement and makes sure the wheelchair is positioned with the rear wheels as close as possible to the kerb. The wheelchair occupant must then lean backwards. This way, the wheelchair will tip backwards and the caretaker can pull the chair onto the pavement. The assistant needs to make sure that the front wheels are on the pavement before letting the wheelchair down.

ⓘ Warning:

Do not drive along the kerbs.

→ Getting down off the pavement:

The assistant needs to move the wheelchair to face the kerb. Then, the occupant has to lean back so the assistant can tilt the wheelchair to find a balance point. When the wheelchair is balanced, the assistant can move the wheelchair slowly off the curb. When the wheelchair is off the pavement, the assistant can place the front wheels back onto the ground.

6.3. Going up and down ramps

We recommend you always examine the slope to see if there are any risks such as holes or irregular surfaces before moving.

ⓘ Warning:

Do not carry any weight on the back of the wheelchair whilst ascending. This changes the balance and increases the chances of tipping backwards.

Remember the recommendation previously given to ensure the footrests are a minimum of 7 cm off the ground. One reason for this is that you need at least this clearance when you are descending a slope in order to ensure the footplates do not catch the end of the slope.

6.4. Going up and down stairs

Stairs are one of the greatest obstacles for wheelchair users. However, you can get up and down stairs in a wheelchair with a little help from two assistants. The assistants must assess the weight to be moved up or down the stairs before any manoeuvring.

→ Going up stairs:

You need to go up the stairs backwards. One assistant should be at the front of the wheelchair and another at the back. The assistant at the back must hold onto the grips to tilt the wheelchair backwards until it is in balance. When the wheels are in balance, the assistant at the front needs to grab the side frames. You are now in a position to go up the stairs. The assistant at the front pushes the side frames up until they are above the first step and the assistant at the back will take the first step on the stairs. Repeat this technique until you get past the last step. You can let the front wheels down when the assistant at the front has taken the last step.

→ Going downstairs:

To go downstairs, the wheelchair should be facing forwards. The assistant in the back should tip the wheelchair until it balances. The assistant at the front should support the wheelchair by holding the side frames. The wheelchair should then be slowly guided down each step.

ⓘ Warning:

Never lift a wheelchair by the grips, armrests or footrests. Always hold a part of the main frame.

7. INSTRUCTIONS FOR USE AND MOUNTING OPTIONS

This section explains all the possibilities and options of your wheelchair. It pragmatically discusses all adjustments and instructions for use such as removing the footrests, etc.

7.1. Technical information

ACWC05#41/45

Seat width	410/450 mm
Seat depth	420 mm
Seat height	540 mm
Backrest height	420 mm
Total width	Seat width + 150 mm
Total length	1080 mm (incl footrest)
Total height	950 mm
Width (folded)	300 mm
Armrest height (adjustable)	160 mm
Total weight	13.80 kg
Wheelchair weight without removable wheels	10.24 kg
Weight of loose wheel	1.79 kg
Weight footrest	0.56 kg
Maximum occupant weight	110 kg
Dimensions to take into account when moving chairs through doors	Seat Width as per the model + 205 mm

7.2. Unfolding and folding the wheelchair

→ Unfolding the wheelchair:

Make sure you stand next to the wheelchair;
Grab both seat tubes and move them apart;
Push both seat tubes downwards so that the wheelchair will unfold as shown in picture 1;

Go to the front of the wheelchair and push the seat tubes towards the side frame. Check to ensure the seat tubes are directly over the side frame and the seat upholstery is flat.

Reposition the seat pad and backrest pad and fold out the footplates.

You can now use the wheelchair.

→ Folding the wheelchair:

Fold away the footplates and remove the footrests;

Remove the seat pad and backrest pad;

Make sure you stand next to the wheelchair;

Place one hand at the front in the middle of the seat upholstery and one hand at the rear side in the middle of the seat upholstery. Pull the seat upholstery straight up, as shown in picture 2.

Image n 1

Image n 2

7.3. Using the footrest.

→ Folding the footplate away:

Go to the front of the wheelchair and grab the front of the footplate, as shown in picture 3.

Fold the footplate as indicated in picture 4.

Image n 3

Image n 4

→ Removing the footrest from the wheelchair:

Stand in front of the wheelchair.

Pull the control lever upwards, as shown in picture 5. You can now swing the footrest inwards or outwards.

Swing the footrests 90° away to the outside of the wheelchair and pull the complete footrest straight upwards from the fixation points, as shown in picture 6.

Image n 5

Image n 6

→ Placing the footrest back on your wheelchair:

Stand in front of the wheelchair.

Reposition the footrest at an angle of 90° back on the wheelchair as seen in picture no. 6.

Swing away the footrest to the front until you hear a click. This click confirms that the footrest is in the correct position.

Check to ensure the footrest is correctly positioned.

→ Adjusting the height of the footrest:

Remove the footrest from the wheelchair.

Turn the footrest with the back side towards you.

Loosen the screw indicated in picture 7.

Move the footrest tube up and down to the desired height.

Tighten the screws firmly.

Make sure the footplate is in place and that the length is equal on both sides.

Image n 7

7.4. Using the brakes

→ Engaging the brake:

Grab the black brake lever.

Push the lever forward, as indicated in picture 8.

You will hear a click when the brake is correctly engaged.

Image n 8

Image n 9

→ Disabling the brake:

Grab the black brake lever.

Pull the lever towards you to disable the brake, as indicated in picture 9.

7.5. Using the armrest

→ Adjusting the height of the armrest:

Push the button to unlock the armrest, as indicated in picture 10 and 11;

Move the armrest up or downwards to the desired height.

Release the button to lock the armrest.

Check to ensure the armrests are correctly locked. If locked correctly, the armrests will remain in the same position when trying to move them up or down.

Image n 10

Image n 11

ⓘ Warning:

You can fold the armrests down when not moving the wheelchair. Otherwise, we recommend you put the armrests in the highest positions while moving in the wheelchair.

7.6. Using the rear wheel

→ Removing the rear wheel

The rear wheels on the wheelchair may be removed by using the 'quick-release' system:

Stand next to the wheelchair.

Tilt the wheelchair away from you.

Press the black button in the middle of the rear wheel, as shown in picture 12 and 13, and pull the rear wheel towards you.

Image n 12

Image n 13

To replace the rear wheel you should follow the above steps in reverse order. Check to ensure the wheels are correctly installed before using the wheelchair.

7.7. Table with wheel models

Type of wheelchair	Front wheel tyre size	Rear wheel tyre size
Self propelled	7" x 1¼" PU	24" x 13/8" PU

7.8. Using the fold-down backrest

→ The folding backrest:

To lower the height of the wheelchair for transporting you can fold down the backrest in the following way:

Stand behind the wheelchair.

Squeeze both black levers at the back of the wheelchair at the same time to fold down the backrest, as shown in picture 14.

Image n 14

Image n 15

The folding mechanism is now unlocked and you can fold down the backrest, as indicated in picture 15.

Make sure the backrest is straight up before sitting in the wheelchair.

You should follow the above steps in reverse order to position the backrest straight up.

7.9. Using the accessories and other wheelchair options

→ Using the storage bag:

There is a storage bag at the back of the wheelchair. To open it, place your hand under the closure and move it upwards, as shown in picture 20.

To close the storage bag, press the closure against the Velcro, as indicated in picture 21.

Image n 16

Image n 17

8. TRANSPORT AND TRANSIT BY CAR

Your wheelchair is designed to be easily transported by car. Your wheelchair is foldable so the total width is limited to an average of 30 cm. Furthermore, your wheelchair is equipped with swing away detachable footrests. At the same time the wheelchair is standard equipped with swing away and removable foot or leg rests. Some wheelchairs may be equipped with a couple of size decreasing accessories like detachable rear wheels.

8.1. De-assembly

Remove the foot- or leg rests on the front side of your wheelchair (see section 7 'Instructions for use and mounting options');

Folding the wheelchair:

1. Stand in front of the wheelchair. Grab the seat from the front and back in the middle and pull up. Tilt the wheelchair to the side and press both sides together to complete the folding.
2. The rear wheels may be removed when necessary when there is a lack of space by following the steps described in section 7, "Instructions for use and mounting options".

8.2. Assembly

If the rear wheels have been removed, you will have to put them back on first. Please follow the instructions in section 7, "Instructions for use and mounting options".

1. When you need to replace the rear wheels, please see section 7, "Instructions for use and mounting options".
2. Stand next to the wheelchair, tilt it a bit towards you and push the seat tube down. The wheelchair will unfold. Stand in front of the wheelchair and push again on the two seat tubes to ensure that they are directly above the frame. You can be sure your wheelchair is in the correct position when the upholstery is flat.

ⓘ Warnings:

Never sit in your wheelchair while being transported.

💡 Tip:

The backrest can be folded to lower the height of the wheelchair for transport.

9. MAINTENANCE

Protect your wheelchair by having it checked regularly. We strongly advise you to have your wheelchair checked at least once a year by a qualified technician and that only original spare parts be used. This periodic inspection will ensure you that the wheelchair will work perfectly for years. There are a lot of things you can do yourself to keep your wheelchair in optimal conditions. Regular checks and constant maintenance will extend the life of your wheelchair and make it easier.

9.1. Warranty

Scope of the warranty

You will receive the factory warranty along with the wheelchair. However, your dealer may offer a supplementary warranty.

This warranty is only granted to you as the consumer. It is not intended to be used commercially (for hiring or institutional use). The warranty is limited to material defects and possible hidden vices.

The wheelchair is covered by a two-year warranty for material or manufacturing defects. Excluded from the warranty are imperfections caused by improper use, accidents or natural wear of parts such as plastic components like the footrests, tyres, inner tubes, upholstery, brakes and similar parts.

If you do not follow instructions, warnings and the notes in the user's manual, you or a third party may be physically injured or the product may be damaged and the warranty may be voided.

Tip:

This warranty is not transferrable.

Replaced or repaired parts are covered by the same warranty conditions as the original wheelchair. Worn parts are normally not guaranteed, unless these parts are worn as a direct result of an original manufacturer defect. These parts include the upholstery, tyres, inner tubes and similar kinds of parts, for example.

The warranty conditions above describe all wheelchair parts, and apply to the models bought at the normal end-user price.

1. Not maintaining the wheelchair and parts according to the manufacturer's instructions or not using the specific original parts;
2. Damaging the wheelchair or parts due to negligence, an accident or improper use;
3. Adjusting the wheelchair or parts other than as indicated by the manufacturer or doing any repairs before notifying the authorised service technician;
4. If the product is not equipped with an original factory frame number and identification label as described in the manual, see section 1.

The wheelchair that is described and presented in this manual may differ from your own model in certain details. However, all instructions are relevant, independent of slightly different details. We reserve the right to change the product in this manual without further notice. All measurements, drawings and capacities indicated in this manual are approximate and may be slightly different to your wheelchair specifications..

Warnings:

ORLIMAN will not be liable for any consequential or individual damage whatsoever. While the purpose of this instructions booklet is to describe proper care, it is not exclusive. You must consult with an authorised dealer if you plan to use the whe-

elchair in any way other than as indicated in this instructions booklet. If any adjustments are made with structural repercussions, the warranty will be completed voided immediately.

Tip:

Then, you will also find extensive information on the maintenance of your wheelchair.

Preventive maintenance is most important and many of these things you can easily do yourself or a friend or family member can help you. The maintenance you can do yourself is highlighted below.

Every month:

- Check how the wheelchair is folding and whether the rear wheels are easy to remove;
- Check the upholstery for wear;
- Check front and rear wheel movement and check how well the wheels turn;
- Check whether the brakes are working correctly;
- Check for any wear on the tread.

Every three months:

- Check all bolts, screws and nuts and tighten if necessary;
- Check all spokes for tension;
- Check whether the brakes are working correctly;

We recommend you have the conditions of your wheelchair checked at your orthopaedic store..

9.2. Wheels

Examine tyres periodically for wear and replace them as needed.

PU tyres: polyurethane (PU) tyres are most often used for wheelchairs. They can be recognised as there is no valve. The advantage to these types of tyres is that they do not need to be pumped up and cannot be punctured. Just as occurs with air tyres, polyurethane (PU) tyres can wear out. Periodically check to ensure the tread is still deep enough.

9.3. Brakes

Inspect both brakes for sharp rims. If there are sharp rims on the brakes we advise you to replace them. Ensure that all the parts of the brake are in the right position and secured.

- Make sure that the mechanism is working smoothly. If it is not working smoothly, apply a few drops of WD40 or Teflon oil to the turning points. Remove any excess or dirt.
- Make sure the brakes do not come into contact with the wheels as you are moving.

9.4. Cross frame

The cross frame of the wheelchair has to be in good condition. The cross frame is the foundation for your wheelchair.

- Make sure the wheelchair can be easily folded and unfolded.
- Make sure the wheelchair travels in a straight line while moving.
- Make sure the cross frame is not bent and shows no signs of wear.

9.5. Rear wheels

Properly adjusting the rear wheels improves the stability of the wheelchair. It will also allow you to use less energy to move the wheelchair.

- Make sure that the wheels can move freely without any friction.
- Make sure there are no irregular edges on the rear wheels.

9.6. Front wheels and front castor forks

The front wheels and front forks must move and turn freely to ensure a smooth ride.

- Make sure the nut on the front fork is not too loose or too tight. When the nut is too loose, the wheels will rotate but the ride will be uncomfortable. If the nut is too tight, the chair will be difficult to steer because the bearings are pushed in such way the balls cannot move freely.
- Make sure the front castor forks show no signs of wear.

9.7. Upholstery

It is important to regularly check the condition of your wheelchair upholstery. Torn or worn upholstery will no longer support human weight and this can create dangerous situations.

Check the textile lining for holes, cracks and worn areas.

Check the upholstery-mounting grommets to make sure that they are all correctly aligned and secure.

9.8. Cleaning

It is important to clean your wheelchair regularly.

- Clean the upholstery, chassis and plastic components on the wheelchair regularly with mild soap and water. Never use abrasive cleaners because they can damage the scratch the wheelchair structure. Also never use steam or high pressure cleaners.
- Wax the varnish of the chassis regularly. Never use solvents, abrasive waxes, caustic chemicals or spray silicone.
- Dry your wheelchair after cleaning. Also dry your wheelchair after you have been using it during a rainstorm.

Tip:

You can also use a car shampoo to clean the wheelchair. This cleans, protects and gives a beautiful gloss to your wheelchair. Sand and sea salt can rust and corrode the wheelchair structure. Clean the chair well after exposure to these factors. If the wheelchair is going to be used by another occupant, all the parts that entered into contact with the prior patient must be cleaned with a disinfectant.

10. TROUBLESHOOTING

Your wheelchair has been inspected and adjusted so that you can use it immediately. Maintenance is required with constant use, specially if the factory settings have been changed.

The following troubleshooting guide lists the most common problems that may occur and offers corrective actions for each.

 **Tip:**

We recommend only using ORLIMAN replacement parts. Otherwise, the warranty will not be valid.

Symptom							
Looseness in chair	Squeaks / rattles	Wheel vibration	The wheelchair is difficult to move	Slower turns	Chair veers left	Chair veers right	
							POSSIBLE CAUSE AND CORRECT SOLUTION
✓	✓	✓		✓			Check to ensure all nuts and bolts are tight. See paragraph 5.9.
		✓		✓	✓	✓	Rear wheels and/or castors may be adjusted improperly. Make sure that both rear wheels and castors are mounted in identical positions.
			✓				Check whether the bearings are worn.
		✓		✓	✓	✓	Castor forks may be adjusted improperly. Make sure they are adjusted properly.

 **ORLIMAN S. L. U.**
C/ Ausias March, 3 - Pol. Ind. La Pobla-L'Eliana
Apdo. de correos 49 · C.P.: 46185
La Pobla de Vallbona Valencia - España (Spain)

MADE IN SPAIN

LBIACWC#2_ES_EN

EDICIÓN/EDITION: 2018-07