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Fecha de revisión/Revision date: 2019-10 | v.01

LBIAO#A15

Dear Customer,
Thank you very much for placing your trust in an Orliman product. Please read the instructions carefully. Keep these instructions and the packaging for future reference. If any serious incidents related to the product occur, notify Orliman S.L.U. and the corresponding competent authority in your country.

SUITABLE FOR

ACE503 / ACE505 / ACE511 / ACE514

- Tendinitis. Arthritis. Sprain. Carpal tunnel syndrome. Instability. Postoperative or post-traumatic recovery. Rhizarthrosis.

ACE504 / ACE506 / ACE507 / ACE510 / ACE512P / ACE513 / ACE515

- Tendinitis. Rhizarthrosis. Sprain. Instability. Postoperative or post-traumatic recovery.

PLACEMENT

In order to ensure a correct product fit, it is essential to choose the correct size (the packaging shows a size guidance chart with centimetre equivalents). Be aware that the garment may come in a universal size).

If the product must be adapted, any such adjustments must be done by an orthopaedic specialist or healthcare professional legally certified to do so who must make sure the end user or person responsible for fitting the product properly understands how it works and should be used. Adjust the product to ensure it permits natural movement and does not cause any irritation or discomfort when used.

Follow the pictures (if given). We advise that you tighten the product sufficiently to achieve a snug fit, without applying excessive pressure.

ACE511

- 1-Insert the hand with the thumb in the corresponding opening.
- 2-Adjust the forearm strap and secure with the micro-hook.
- 3-Move the wrist strap to the desired position and tighten the double strap for the desired pressure on the wrist and hand.
- 4-Close the 2 palm straps; the tension and direction can be changed.

ACE512P

- 1-Mould the thumb part in accordance with the perimeter of the digit and the 2 base strips for this part so it fits the wrist shape well.
- 2-Put the wrist brace on according to the instructions for ACE511 (see above).
- 3-Put the thumb piece on by putting the digit in the desired position.

ACE507 / ACE515

- 1-Put the wrist brace on and adjust the thumb splint to the perimeter of the digit.
- 2-Close the wrist strap and then place the thumb in the metal ring, securing it with the respective strap.

ACE510

- 1-Insert the thumb in the opening designed for this digit.
- 2-Wrap the pressure strap around the wrist.
- 3-Adjust the elastic strap until comfortable and secure with the Velcro®.
- 4-If necessary, mould the aluminium splint for the desired position.

ACE503 / ACE504 / ACE505 / ACE506 / ACE514 / ACE513

- 1-Insert the wrist brace and position the palm splint in the palm of the hand being careful to align it with the wrist axis.
- 2-Adjust the forearm and hand strap(s) to the desired tension and secure with the quick closure.
- 3-Adjust the strap for the thumb (as applicable) to the desired tension and secure with the quick closure.

RECOMMENDED CARE INSTRUCTIONS

Hand wash from time to time using warm water, pH neutral soap, and allow to dry naturally. Store in a dry place at room temperature.

**WARNING**

Do not use this product on an open wound. Should irritation occur, stop using the product and inform your doctor. Do not place near heat sources and do not combine with other products such as creams or ointments.

Products marked with the symbol contain natural rubber latex and can cause allergic reactions in people sensitive to latex.

Products marked with the (Ref.: ACE507, ACE511, ACE512) symbol contain ferromagnetic components and, therefore, extreme precaution must be taken if you undergo an MRI scan or are exposed to radiation associated with diagnostic or therapeutic procedures.

KEEP OUT OF THE REACH OF CHILDREN.

This article is defined as a class I medical device.

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