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Dear Customer,
Thank you very much for placing your trust in an Orliman product. Please read the instructions carefully. Keep these instructions and the packaging for future reference. If any serious incidents related to the product occur, notify Orliman S.L.U. and the corresponding competent authority in your country.

SUITABLE FOR

ACE518 / ACE520

Carpal tunnel syndrome, tendinitis, tendovaginitis, (inflammation of the tendon sheath), tenosynovitis, traumas that affect the triangular ligament of the wrist, as a post-operative and sport activity precautionary measure. Stabilising system of the lower radial and cubital joint.

ACE519

Arthrosis of the trapezium-metacarpal joint, cubital ligament injuries, metacarpophalangeal of the thumb, Post-operative care, Bennet fracture, surgical repair of ligament injuries.

In cases where it incorporates both splints (palm and thumb), consider both suitabilities as above.

FITTING INSTRUCTIONS

For best therapeutic results considering different pathologies and to extend the useful life of the product, it is essential to choose the correct size for each patient or user. Excessive compression may be intolerable; adjusting the compression to be firm yet comfortable is recommended.

If the product must be adapted, any such adjustments must be done by an orthopaedic specialist or healthcare professional legally certified to do so who must make sure the end user or person responsible for fitting the product properly understands how it works and should be used.

ACE518 / ACE520

Shape the palm splint (Cock-up splint), achieving the desired palm or dorsal flexion.

Open the dorsal facing velcros of the orthosis. Place hand on the inside of the orthosis allowing it to meet the splint, with the first corner of the thumb and the distal edge right behind the metacarpophalangeal joint.

If also incorporating the thumb splint:

Insert thumb allowing for flexion, at interphalangeal distal. The metallic spring can be adjusted so that the thumb is located in front of the second finger (index finger), at an angle of approximately 30°. The distal edge on the palm face coinciding with the metacarpophalangeal joint. Firmly adjust the velcros avoiding any disorder to the circulation.

ACE519

Insert thumb, allowing for flexion at distal interphalangeal joints, if necessary trim the plastic.

The metallic spring can be adjusted so that the thumb is located in front of the second finger (index finger), at an angle of approximately 30°.

- 1-Insert thumb allowing for flexibility, at distal interphalangeal joints, and the distal edge on the palm face coinciding with the metacarpophalangeal joint
- 2-Firmly adjust the velcros avoiding any disorder to the circulation.

⚠ PRECAUTIONS

Before each use, check that all product components are present as per the fitting process. Periodically check the conditions of the product. If you observe any defect or anomaly, immediately report it to the issuing establishment.

This product is made of inflammable material. Do not expose the products to situations that could set them on fire. In the event of a fire, quickly get them off your body and use the proper resources to extinguish the fire.

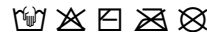
To avoid minor discomfort caused by sweating, we recommend using some type of cotton fabric to separate the skin from contact with the product material. For discomfort such as chafing, irritation and swelling, remove the product and see a doctor or orthopaedic specialist. The product should only be used on healthy skin. It is not recommended for use over open scars with swelling, redness or hotspots.

Products marked with the  (ACE518, ACE519, ACE520) symbol contain natural rubber latex and can cause allergic reactions in people sensitive to latex.

Products marked with the  (ACE518, ACE519, ACE520) symbol contain ferromagnetic components and, therefore, extreme precaution must be taken if you undergo an MRI scan or are exposed to radiation associated with diagnostic or therapeutic procedures.

RECOMMENDED CARE INSTRUCTIONS

Hand wash from time to time using warm water, pH neutral soap, and allow to dry naturally. Store in a dry place at room temperature.

KEEP OUT OF THE REACH OF CHILDREN.

MD This article is defined as a class I medical device.

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