Español

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English

CONTENTS / DESCRIPTION

Lilaë Move® maternity tights:
- 1 pair of tights.

Product leaflet to read carefully.

INDICATIONS

Tired legs with slight pain or swelling. Problems of small varicose veins or vascular spiders. Prevention of venous pathologies.

COMPOSITION

81% polyamide, 19% elastane.

SAFETY INSTRUCTIONS

If rash, irritation or swelling remove the product and consult a doctor or prosthetist. Protect any part of the body to which an analgesic may have been applied before fitting the product. The contact of the product with cream or body oil may deteriorate the quality of compression. It is not recommended for other patients to reuse this product. In the event of a serious incident with the device, inform the manufacturer and the competent authority.

CONTRAINDICATIONS

Allergy to the product components

WASHING INSTRUCTIONS

- Wash by hand with neutral soap (max. 30 °C).
- Rinse carefully.
- Allow to dry in the open air.
- Do not dry in a dryer.
- Do not use bleach.
- Do not expose these products to situations where they could ignite.

Store at room temperature.

In order to dispose of the packaging and the product, strictly comply with the legal norms of your community.

FITTING

You are recommended to put on the tights in the morning as you get up.

- Turn over the tights and push the ends inwards.

- Put on the tights carefully starting with the toes followed by the heel.

- Slide the tights up to the knees on both legs.

- Finish by standing up and pulling them slowly upwards to the abdomen so that they are well stretched.

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