Ref.: OSL1300 |

Ref.: OSL1301 |

Patuco antiescaras Soft
 Soft pressure relief heel protector
 Chaussons anti-escarres Soft
 Druckentlastungs-Fersenpolster Soft
 Protetor calcanhar antiescaras Soft

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ENGLISH

INSTRUCTIONS FOR USE AND PRESERVATION

Dear customer,

Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value. Please, read the instructions carefully. If you have any doubt, get in contact with your doctor, your orthopaedist specialist or our customer service department. Orliman appreciates your choice and wishes you a speedy recovery.

REGULATION

These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimising all the existing risks. The trials have been carried out in accordance with the European regulation UNE-EN ISO 22523 of Prosthesis and Orthesis.

SUITABLE FOR

- Prevention and auxiliary treatment of pressure sores.
- People who spend long periods of time in the same position with parts of the body in contact with the surface of the bed or subjected to constant friction by bedding or rough surfaces.

Protection areas by product:

- Boot: calcaneus (heel) and malleoli.
- Elbow protector: olecranon, epicondyle and epitrochlea.

FITTING INSTRUCTIONS

In order to achieve the highest degree of therapeutic effectiveness for different disorders and to prolong the service life of the product, it is essential to select the most suitable size for the patient or user (see the size and centimetre equivalence chart on the packaging). As excessive compression can cause intolerance, it is advisable to adjust the compression to an optimum level.

Follow the fitting instructions below:

1-Undo the product's adjustable straps or fastenings if not fit to the appropriate part of the body.
2-Fasten for a comfortable fit.

WASHING AND MAINTENANCE RECOMMENDATIONS
Maintenance (SOFT heel protector and SOFT elbow protector): regularly brush or shake the surface of the fibre to keep it in optimum condition and maximise the product's service life.

Maintenance (TECH heel protector): wipe the outer fabric with a damp cloth. Use a dry towel to absorb any moisture.
Washing: before washing, close any Velcro fasteners or zips.
Machine washable: machine wash with delicate programme and tepid water (30°C) with neutral detergent. Spin dry gently (400 rpm). Leave to dry at room temperature. Do not use bleach. Do not hang up, iron or expose to direct heat sources such as stoves, heaters, radiators, direct sunlight, etc. Do not dry clean.

When using or cleaning, do not use bleach, alcohol, ointments or liquid solvents. If the product is not well wrung out, detergent residue can remain and cause irritation to the skin and damage to the product.

Store in a dry place at room temperature.

GUARANTEES

ORLIMAN, S.L.U. Guarantees all its products, as long as these have not been manipulated or altered from their original state. Does not guarantee those products in which as a result of misuse, deficiencies or breakage of any kind, its original state has been altered. If you observe any deficiency or anomaly, communicate this immediately to the establishment from which it was obtained in order for it to be changed.

ORLIMAN, S.L.U. Appreciates your choice and wishes you a speedy recovery.

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