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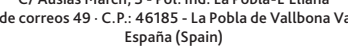
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
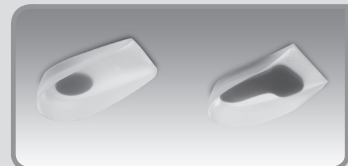
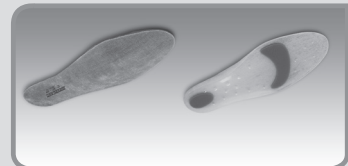
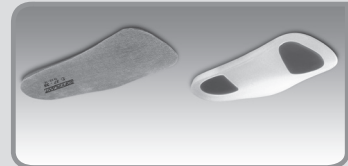
English

INSTRUCTIONS FOR USE AND PRESERVATION Dear customer, Thank you for your trust in this Orliman product. You have acquired a quality product with a high medicinal value...

REGULATION These products satisfy the European Medical Device Directive 93/42/CEE (RD. 1591/2009). A Risk Analysis has been carried out (UNE EN ISO 14971) minimizing all the existing risks...

SUITABLE FOR Heel cups: Suitable for relieving talalgias, bursitis of the heel and tendinitis of the Achilles tendon...

Insoles: Talalgia, bursitis, metatarsalgia, Calcaneus spur, diabetic foot, plantar fasciitis, painful or tired feet, sporting practice complaints...

FITTING INSTRUCTIONS THE PRODUCT SHOULD BE ADJUSTED BY YOUR ORTHOPAEDIC TECHNICIAN OR IN ACCORDANCE WITH DOCTOR'S ORDERS...

In order to obtain a greater degree of therapeutic effectiveness in the different pathologies and to prolong the life and utility of the product...

To fit observe the following aspects:

- 1- Choose the appropriate size. 2- Place a heel cup or an insole in each foot, unless medically prescribed.

PRECAUTIONS The constructive material is inflammable. Do not expose these products to situations where they could ignite...

tor or prosthesis. Contraindicated in open scars with swelling, redness and accumulation heat.

RECOMMENDATION- WARNINGS These products are to be used only by those which have been medically prescribed to do so. Persons unconnected to such prescriptions should not use them...

VORSCHRIFTEN The use of the orthesis is conditioned to the recommendations of the prescribing doctor, therefore it must not be used for any other means than those prescribed as the guarantee...

MANUFACTURE - CHARACTERISTICS The material features used in production have been tested and approved and meet all the European quality requirements...

RECOMMENDATIONS FOR PRESERVATION AND CLEANING Periodically wash by hand with lukewarm water and a neutral soap...

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GUARANTEES ORLIMAN, S.L.U. guarantees all its products, as long as these have not been manipulated or altered from their original state...

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